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EN - Ready-to-use disinfectant for impressions

Zeta 7 Spray is an aldehyde-free, ready-to-use disinfectant with a wide range of action, prepared and tested in compliance with new harmonised European disinfection regulations. Zeta 7 Spray is designed for rapid, high-level disinfection of impressions (addition and condensation), in silicone, alginate, polyether and polysulphur: the impressions remain unaltered, have no significant dimensional change and are compatible with gypsums. The Zeta 7 Spray improves gypsum smoothness on impression surfaces, reduces the possible formation of bubbles and allows for a more faithful reproduction of detail.

Activité virucide:	3 minutes	conditions de saleté (EN 14476)
Activité (test en suspension):	1 minute	bactéricide (EN13727)
	1 minute	fungicide (EN 13624, test sur Candida albicans)
	2 minutes	tuberculocide (EN14348)
Activité (test de surface):	2 minutes	tuberculocide (EN14563)

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Warnings, precautions and storage conditions:
This product has been designed for use by qualified operators with the knowledge of safety regulations. Use protective gloves. Operate in well-ventilated areas. Store the closed container in a cool dry place. Avoid contact of Zeta 7 Spray with alcohol-sensitive materials. Avoid contact of Zeta 7 Spray with other disinfectants or cleaners. Keep away from heat/sparks/open flames/hot surfaces. No smoking.

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