

elite[®] glass



Clinical

A-Silicone Clear Matrix Material

Zhermack[®]
CLINICAL





elite[®] glass

The clarity of detail

ELITE[®] GLASS is the new A-silicone from **ZHERMACK** in 50 ml automix cartridges for the construction of extremely transparent matrices, used for the direct processing of **cosmetic restorations**.

OPERATING VERSATILITY

ELITE[®] GLASS can be used in a **variety of clinical situations**, processing the matrix directly from the mouth of the patient (e.g. pigmentations) or alternatively indirectly, based on a model (e.g. fixing gaps, correcting rotations, fractures, dental anomalies, temporary crowns).

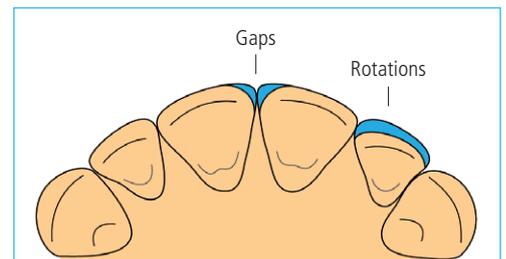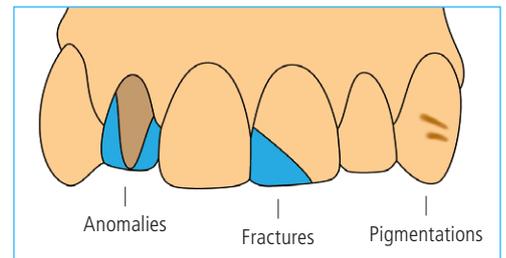

EXTREME TRANSPARENCY

The new formulation of **ELITE[®] GLASS** ensures **highly transparent** matrices, guaranteeing the **perfect photopolymerisation** of the composite used for the restoration.

IDEAL RIGIDITY

The ideal rigidity of the matrices created using **ELITE[®] GLASS** ensures **the original shape** remains unaltered, and simplifies repositioning **without deformations**.

QUALITY AESTHETIC RESULTS

The matrices created using **ELITE[®] GLASS** allow *excellent aesthetic results* to be achieved in a *short time*, simplifying the clinical procedures.


DIRECT CLINICAL CASE



Initial case



Creation of the matrix with ELITE[®] GLASS: leave at least 2 mm of silicone around teeth*

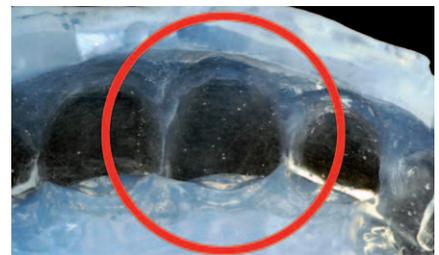

Make sure there are no defects in the area involved in the impression



Preparation of the veneer



Treatment of the surface**



Application of the composite, starting from the incisor zone and moving towards the cervical edge



Repositioning the matrix made with ELITE[®] GLASS in the mouth, making sure not to leave gaps between the matrix and the teeth. Photopolymerise through the matrix**



Finishing the composite: remove the excess material and polish**



End result

*Important: remember to trim the level of material at least when first used
**Follow the instructions provided by the manufacturer of the composite

TECHNICAL DATA

Working time	40"
Time in mouth	1'30"
Setting time	2'10"

The working time is intended from the beginning of the mixing at 23°C (73°F), the time in mouth is intended at 35°C (95°F)



Packaging

C401610 ELITE GLASS	2 cartridges (Base+Catalyst), 50 ml each + 6 mixing tips, green
C401611 ELITE GLASS INTRO KIT	1 cartridge (Base+Catalyst), 50 ml + 3 mixing tips green + 1 partial upper hi-tray light clear

Accessories

C202080 MIXING TIPS MEDIUM/GREEN	Mixing tips green (48 pcs)
C202100 DISPENSER D2	D2 dispenser - 1:1
C207203 PLUNGER	Plunger for D2 dispenser - 1:1
D5CPUP HI TRAY LIGHT CLEAR	Partial upper hi-tray light clear (12 pcs)

BRANCHES

GERMANY
Zhermack GmbH
Deutschland
Öhlmühle 10 - D-49448 Marl
Tel. +49 - 0 54 43 / 20 33 - 0
Fax +49 - 0 54 43 / 20 33 - 11
info@zhermack.de
www.zhermack.com

POLAND
Zhermapol Sp. z o.o.
ul. Augustówka 14
02 - 981 Warszawa - Polska
Tel. +48 - 22 858 82 72
+48 - 22 858 73 41
Fax +48 - 22 642 07 14
biuro@zhermapol.pl
www.zhermapol.pl

USA
Zhermack Inc.
P.O. Box 4195, River Edge
New Jersey 07661 - 4195
Tel. +1 - (877) 819 6206
+1 - (732) 389 8540
Fax +1 - (732) 389 8543
info@zhermackusa.com
www.zhermackusa.com

COUNTRY CONTACTS

UK AND IRELAND
Tel. +44 - (0)7870 690 811
uk@zhermack.com

SPAIN
Tel. +34 - 900 99 39 52
info.spain@zhermack.com

FRANCE
Tel. +33 - 0800 915 083
info.france@zhermack.com

RUSSIA
Tel. +7 - 916 930 1191
info.russia@zhermack.com

MEXICO AND CENTRAL AMERICA
Tel. +52 - 1 55 919 804 97
info.mexico@zhermack.com

SOUTH AMERICA
Tel. +57 - 310 2601 777
info.colombia@zhermack.com

Zhermack SpA

Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY
Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com